PROSTATE-PSA TEST TESTE DA PRÓSTATA-PSA TEST DE PRÓSTATA-PSA

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Cut-off: 4 ng/mL

N P

Self-Test for the detection of the Prostate Specific Antigen - PSA

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PROSTATE AND PSA

In man, the prostate antigen (PSA) is secreted by the prostate. This gland, situated in the abdomen under the bladder, surrounds the initial part of the urethra and plays an important role in the semen.

The prostate antigen level gives information about the prostate physiological condition. Thus, levels above the norm may show pathology of the prostate (benign hypertrophy, prostatitis, cancer, etc.).

The PROSTATE-PSA TEST should be performed at least once a year in men over 50 years. In case of prostate problem, early screening allows considerably increasing recovery possibilities.

CONTENT


THE BOX CONTAINS ALL THE NECESSARY TO PERFORM 1 PSA TEST:

- 1 sealed aluminium pouch containing:


1 plastic pipette
1 desiccant bag


1 Prostate PSA Test cassette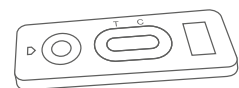

- 2 sterile lancets for blood sampling

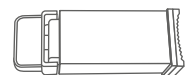- 1 disinfectant pad
- 1 instructions leaflet

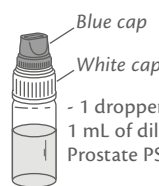Blue cap
White cap
- 1 dropper bottle containing 1 mL of diluent Prostate PSA Test

* Not provided necessary material: absorbent cotton.
* Only open the protective pouch when you are ready to use the test.
* The desiccant bag should not be used.

PRECAUTIONS

- Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
- Keep out of the reach of children.
- Do not use after the expiry date printed on the label and on the protective pouch or if the pouch is damaged.
- Follow strictly the indicated time, blood and diluent quantities.
- Store between +4 °C and +30 °C. Do not freeze.
- Do not re-use the cassette and lancet.
- External use only. DO NOT SWALLOW.
- This test is exclusively intended to in vitro diagnostic self-testing use.
- Not recommended for use by individuals taking medicines that thin the blood (anticoagulants) or by the individuals with disorders such as hemophilia.
- After use, dispose of all components in accordance with local regulations, seek advice from your pharmacist.

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






REF 100080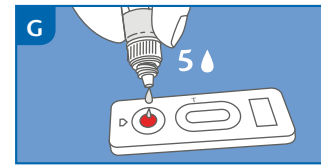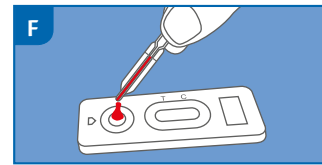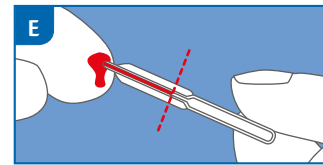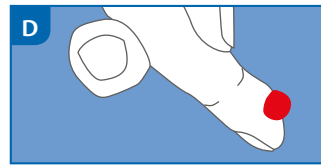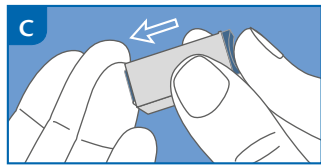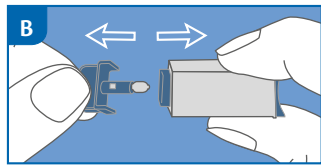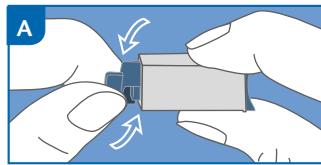

PROCEDURE

- 1) Wash your hands with soap and rinse with clear warm water.
- 2) Tear the protective pouch (from the notch) and only get out the cassette and the pipette. Dispose of the small desiccant bag.
- 3) Hold the lancet and carefully rotate 360° the protective cap without pulling it. - **FIG. A**
- 4) Extract and dispose the released cap. - **FIG. B**
- 5) Press the lancet, on the side from where the cap was extracted, against the fingertip (it is advisable the ring finger side). - **FIG. C** The tip retracts automatically and safely after use.
- 6) Keeping the hand down massage the end that was pricked to obtain a blood drop. - **FIG. D**
- 7) **Without pressing the pipette bulb**, put it in contact with the drop. The blood migrates into the pipette through the capillarity to the line indicated on the pipette. - **FIG. E**
You may massage again your finger to obtain more blood if the line is not reached. As far as possible, avoid of air bubbles.
- 8) Put the blood collected with the pipette into the sample well of the cassette, by pressing on the pipette bulb. - **FIG. F**
- 9) Wait for the blood to be totally dispensed in the well. **Unscrew the blue cap** of the plastic dropper (leave the white cap tightly screwed) and add **5 drops** of diluent into the sample well of the cassette. - **FIG. G**
- 10) Read the result after **10 minutes**.

RESULT INTERPRETATION


**READ THE RESULT AFTER 10 MINUTES.
DO NOT READ THE RESULT AFTER 15 MINUTES.**

The intensity of the line colour does not have any importance for the interpretation of the test result.


POSITIVE RESULT

Duas coloured lines appear in the window under the marks T (Test) and C (Control). The intensity of the line T may be clearer than the intensity of the line C. This result means that the prostate specific antigen level is higher than the norm and that you should consult a physician.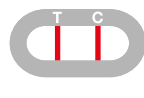


NEGATIVE RESULT

Only one coloured line appears under the mark C (Control). This result means that the prostate specific antigen level is normal. A negative result cannot rule out the possibility of a prostate cancer.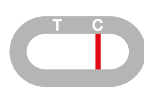

NON VALID RESULT

No line appears or a coloured line appears under the mark T (Test) without any line under the mark C. In this case, it is not possible to interpret the test, which must be considered as non valid. It is recommended to repeat the test with a new **PROSTATE-PSA TEST** and a sample of fresh blood.

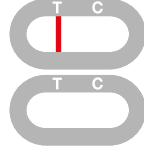F.A.Q. - QUESTIONS & ANSWERS

HOW DOES THE PROSTATE-PSA TEST WORK? The prostate specific antigen is a protein secreted in the semen by the prostate. A part of this protein is found in the blood. The blood PSA level allows estimating the prostate physiological condition. A value higher than the norm indicates prostate hypertrophy. This hypertrophy may be benign, be due to prostatitis or may be cancerous in case of adenocarcinoma. Screening of prostate pathologies is for men over 50 years and is performed up to 75 years. The **PROSTATE-PSA TEST** allows determining a high blood level, above 4 ng/mL, of prostate specific antigen likely to show benign or malignant prostate hypertrophy.

WHEN SHOULD THE TEST BE USED? The test can be performed anytime of the day. However, in the following cases, it is recommended to wait the indicated time before using the test to avoid false positive results:

	Waiting time
Cycling/Ergometry	24 hours
Ejaculation	24 hours
Prostate massage	2-3 days
Transrectal ultrasounds	2-3 days
Cystoscopy	1 week
Prostate transurethral resection/biopsy	4-6 weeks

CAN THE RESULT BE INCORRECT? The results are accurate as far as the instructions are carefully respected. Nevertheless, the result can be incorrect if the **PROSTATE-PSA TEST** gets wet before test performing or if the quantity of blood dispensed in the sample well is not sufficient, or if the number of diluent drops are less than 4 or more than 5. The plastic pipette provided in the box allows making sure the collected blood volume is correct.

HOW TO INTERPRET THE TEST IF THE COLOUR AND THE INTENSITY OF THE LINES ARE DIFFERENT? The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and clearly visible. The test should be considered as positive whatever the colour intensity of the test line is.

WHAT IS THE LINE THAT APPEARS UNDER THE MARK C (CONTROL) FOR? When this line appears, it only means that the test was performing well.

IF I READ THE RESULT AFTER 15 MINUTES, WILL THE RESULT BE RELIABLE? No. The result should be read within 10 minutes after adding the diluent. The result is reliable up to 15 minutes.

WHAT DO I HAVE TO DO IF THE RESULT IS POSITIVE? If the result is positive, it means that the PSA level present in your blood is higher than the norm (4 ng/mL) and that you should consult a physician to show the test results. Then, the physician will decide what you have to do.

WHAT DO I HAVE TO DO IF THE RESULT IS NEGATIVE? If the result is negative, it means that the PSA level in your blood is below 4 ng/mL. For the patients aged between 50 and 75 years or having relatives suffering from prostate cancer, it is recommended to perform a test every year.

WHAT IS THE ACCURACY OF THE PROSTATE-PSA TEST? The **PROSTATE-PSA TEST** is very accurate and has been used for more than 10 years by professionals (hospitals, laboratories, ...) in the field. Evaluation reports show an overall agreement of at least 88% with reference methods.

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